




Skin and Wound Product Information Sheet

Adaptic	
Classification	Layer: Petrolatum Impregnated
Key Points	<ul style="list-style-type: none"> • Petrolatum impregnated cellulose acetate dressing
Indications	<ul style="list-style-type: none"> • For prevention of wound bed trauma by decreasing adherence of the secondary dressing
Precautions	<ul style="list-style-type: none"> • Avoid using this product with silicone-type products as they have the same functionality
Contraindications	<ul style="list-style-type: none"> • Do not use for clients with sensitivity or allergy to petrolatum • Do not use during Hyperbaric Oxygen therapy procedure
Formats & Sizes	<ul style="list-style-type: none"> • Sheet <ul style="list-style-type: none"> ▪ 7.6 x 7.6 cm ▪ 7.6 x 20.3 cm ▪ 7.6 x 40.6 cm ▪ 12.7 x 22.9 cm <div style="text-align: right; margin-top: 10px;">  </div>
Application Directions	Rationale
Cleanse/irrigate wound with sterile normal saline or agency approved wound cleanser; dry peri-wound skin.	Reduces wound debris and allows for adhesion of dressing or tape.
If required, apply skin barrier to peri-wound skin.	To protect the peri-wound skin from maceration and to improve the adhesion of the dressing or tape.
To Apply	
Adaptic can be cut to size. Choose a size that covers the wound plus at least 2 cm of peri-wound skin.	To ensure adequate adhesion to peri-wound skin.
Apply Adaptic in a single layer over the wound and smooth onto the surrounding skin to get a good seal.	Single layer allows passage of exudate through to cover dressing.
Apply appropriate cover dressing to maintain a moisture-balanced wound environment.	The choice of the cover dressing will depend upon the amount of exudate expected.
To Remove	
Carefully lift away the dressing and discard.	To avoid trauma to the wound bed.
Frequency of Dressing Change	
Will depend upon the amount of exudate. Adaptic may be left in place for up to 7 days.	
Expected Outcome	
Contact layer does not adhere to wound.	
For further information, please contact your Wound Clinician.	