




Skin and Wound Product Information Sheet

Allevyn Gentle	
Classification	Cover Dressing: Foam with Silicone
Key Points	<ul style="list-style-type: none"> Combines an absorbent hydrocellular pad sandwiched between a soft gel layer Coated with silicone adhesive layer for atraumatic dressing removal Outer pink side is shower-proof, bacteria proof and prevents strike-through of exudate
Indications	<ul style="list-style-type: none"> For wounds with moderate to large amount of exudate
Precautions	<ul style="list-style-type: none"> N/A
Contraindications	<ul style="list-style-type: none"> Do not use with oxidizing agents such as hypochlorite solutions (Eusol) or hydrogen peroxide Do not use if redness or sensitivity occur
Formats & Sizes	<ul style="list-style-type: none"> ▪ Non border dressing <ul style="list-style-type: none"> ▪ 5 x 5 cm ▪ 10 x 10 cm ▪ 10 x 20 cm ▪ 15 x 15 cm ▪ 20 x 20 cm <div style="text-align: right; margin-top: 10px;">  </div>
Application Directions	
Cleanse/irrigate wound with sterile normal saline or agency approved wound cleanser; dry peri-wound skin.	Reduces wound debris and allows for adhesion of dressing or tape.
If required, apply a skin barrier to peri-wound skin.	To protect the peri-wound skin from maceration and to improve the adhesion of the dressing or tape.
To Apply	
Choose a dressing size that will ensure that the dressing extends at least 2cm beyond wound edge.	Incorrect sizing will adversely affect the dressing absorption functionality.
Apply directly on the wound as a primary dressing or in combination with another wound product as a secondary cover dressing.	Needs to be in contact with the wound to be most effective.
Place the dressing directly over the center of the wound ensuring the white side is applied to the wound.	
The dressing may be notched to improve application over curves.	
Tape edges securely.	Do not cover with a transparent film as this will decrease the wear time of the dressing and increase the possibility of maceration.
To Remove	
Gently lift the border or tape to remove the dressing.	To minimize trauma to the peri-wound skin.
Frequency of Dressing Change	
Will depend upon the amount of exudate. Change when exudate extends to within 2cm of the edge of the dressing. Can be left on up to 7 days.	The absorbed exudate is clearly visible through the pink backing of the dressing
Expected Outcome	
Exudate is managed with no peri-wound skin maceration.	
For further information, please contact your Wound Clinician.	