




## Skin and Wound Product Information Sheet

<b>Allevyn Gentle Border Lite</b>	
<b>Classification</b>	<b>Cover Dressing: Foam with Silicone</b>
<b>Key Points</b>	<ul style="list-style-type: none"> <li>Coated with silicone adhesive layer for atraumatic dressing removal</li> <li>Outer pink side is showerproof, bacteria proof and prevents strike through of drainage</li> </ul>
<b>Indications</b>	<ul style="list-style-type: none"> <li>For wounds with small amount of exudate</li> </ul>
<b>Precautions</b>	<ul style="list-style-type: none"> <li>N/A</li> </ul>
<b>Contraindications</b>	<ul style="list-style-type: none"> <li>Do not use Allevyn Gentle dressings with oxidizing agents such as hypochlorite solutions (Eusol) or hydrogen peroxide</li> <li>Do not use if redness or sensitivity occur</li> </ul>
<b>Formats &amp; Sizes</b>	<ul style="list-style-type: none"> <li>Adhesive border dressing                             <ul style="list-style-type: none"> <li>5 x 5cm</li> <li>5.5 x 12 cm</li> <li>7.5 x 7.5 cm</li> <li>10 x 10 cm</li> <li>15 x 15 cm</li> </ul> </li> </ul> <div style="text-align: right;">  </div>
<b>Application Directions</b>	
Cleanse/irrigate wound with sterile normal saline or agency approved wound cleanser; dry peri-wound skin.	Reduces wound debris and allows for adhesion of dressing or tape
If required, apply a skin sealant to surrounding skin.	To protect the peri-wound skin from maceration and to improve the adhesion of the dressing or tape.
<b>To Apply</b>	
Choose a dressing size that will ensure that the dressing extends at least 2cm beyond wound edge	Incorrect sizing will adversely affect the dressing absorption functionality.
Apply directly on the wound as a primary dressing or in combination with another wound product as a secondary dressing.	Needs to be in contact with the wound to be most effective.
Place the dressing directly over the center of the wound ensuring the white side is applied to the wound.	
<b>To Remove</b>	
Gently lift the border to remove the dressing.	To minimize trauma to the peri-wound skin.
<b>Frequency of Dressing Change</b>	
Can be left on up to 7 days depending on the amount of exudate. Change dressing when exudate extends to within 2cm of the edge of the dressing.	The absorbed drainage is clearly visible through the pink backing of the dressing.
<b>Expected Outcome</b>	
Exudate is managed with no peri-wound skin maceration.	
<b>For further information, please contact your Wound Clinician.</b>	