




Skin and Wound Product Information Sheet

Allewyn Plus Adhesive	
Classification	Cover Dressing: Foam
Key Points	<ul style="list-style-type: none"> • A polyurethane foam with an outer pink side that is shower-proof, bacteria proof and prevents strike-through of exudate
Indications	<ul style="list-style-type: none"> • For wounds with large to copious amounts of exudate • May be used under compression therapy
Precautions	<ul style="list-style-type: none"> • N/A
Contraindications	<ul style="list-style-type: none"> • Do not use for clients with known sensitivity or allergy to polyurethane • Do not use with oxidizing agents such as hypochlorite solutions (Eusol) or hydrogen peroxide • Do not use if redness or sensitivity occur
Formats & Sizes	<ul style="list-style-type: none"> ▪ Adhesive dressing <ul style="list-style-type: none"> ▪ 12.5 x 12.5 cm ▪ 12.5 x 22.5 cm ▪ 17.5 x 17.5 cm ▪ 17 x 17 cm (sacrum) ▪ 22 x 22 cm (sacrum) <div style="text-align: right; margin-top: 10px;">  </div>
Application Directions	
Cleanse/irrigate wound with sterile normal saline or agency approved wound cleanser; dry peri-wound skin.	Reduces wound debris and allows for adhesion of dressing or tape.
If required, apply a skin barrier to peri-wound skin.	To protect the peri-wound skin from maceration and to improve the adhesion of the dressing or tape.
To Apply	
Choose a dressing size that will ensure that the dressing extends at least 2cm beyond wound edge.	Incorrect sizing will adversely affect the dressing absorption functionality.
If using cavity dressing, choose a size that matches the size of the wound cavity. Dressing may be cut as needed,	
Place the dressing directly over the center of the wound ensuring the white side is applied to the wound.	Needs to be in contact with the wound to be most effective.
Apply directly on the wound as a primary dressing or in combination with another wound product as a secondary cover dressing.	
For bordered dressing, smooth borders onto peri-wound skin.	Choice of cover dressing will depend upon the amount of exudate expected.
For cavity dressing, cover with appropriate cover dressing to maintain a moisture-balanced wound environment.	
To Remove	
Gently lift the border to remove the dressing. For cavity dressing, use forceps to gently remove dressing	To minimize trauma to the wound and peri-wound skin.
Frequency of Dressing Change	
Will depend upon the amount of exudate. Change when exudate extends to within 2cm of the edge of the dressing. Can be left on up to 7 days; sacral dressings can be left on up to 5 days.	The absorbed exudate is clearly visible through the pink backing of the dressing
Expected Outcome	
Exudate is managed with no peri-wound skin maceration.	
For further information, please contact your Wound Clinician.	