




Skin and Wound Product Information Sheet

Allevyn Thin	
Classification	Cover Dressing: Foam
Key Points	<ul style="list-style-type: none"> • Dressing is comprised of a polyurethane matrix with a polyurethane top layer • Dressing is self adhesive and waterproof
Indications	<ul style="list-style-type: none"> • For granulating wounds with small amounts of exudate
Precautions	<ul style="list-style-type: none"> • N/A
Contraindications	<ul style="list-style-type: none"> • Do not use for clients with known sensitivity or allergy to polyurethane • Do not use with oxidizing agents such as hypochlorite solutions (Eusol) or hydrogen peroxide • Do not use if redness or sensitivity occur
Formats & Sizes	<ul style="list-style-type: none"> ▪ Adhesive dressing <ul style="list-style-type: none"> ▪ 5 x 6 cm ▪ 10 x 10 cm ▪ 15 x 20 cm <div style="text-align: right;">  </div>
Application Directions	
Cleanse/irrigate wound with sterile normal saline or agency approved wound cleanser; dry peri-wound skin.	Reduces wound debris and allows for adhesion of dressing or tape.
If required, apply a skin barrier to peri-wound skin.	To protect the peri-wound skin from maceration and to improve the adhesion of the dressing or tape.
To Apply	
Choose a dressing size that will ensure that the dressing extends at least 2cm beyond wound edge.	Incorrect sizing will adversely affect the dressing absorption functionality.
Place the dressing directly over the center of the wound. Smooth border onto peri-wound skin.	To optimize the effectiveness of the dressing.
To Remove	
Gently lift an edge to remove the dressing.	To minimize trauma to the peri-wound skin.
Frequency of Dressing Change	
Will depend upon the amount of exudate. Change when exudate extends to within 2cm of the edge of the dressing. Can be left on up to 7 days.	
Expected Outcome	
Exudate is managed with no peri-wound skin maceration.	
For further information, please contact your Wound Clinician.	