




Skin and Wound Product Information Sheet

Cavilon Durable Barrier Cream	
Classification	Skin Barrier: Cream
Key Points	<ul style="list-style-type: none"> A concentrated, therapeutic cream that provides durable, long-lasting protection from body fluids while moisturizing the skin. Resists wash off therefore requiring less frequent applications. Fragrance free.
Indications	<ul style="list-style-type: none"> To protect intact skin and at-risk skin from body fluids including wound exudates and incontinence. To provide moisture to dry skin. Assists with adhesion of tapes and dressings.
Precautions	<ul style="list-style-type: none"> Use with caution on fragile skin as product may increase the adherence of some adhesive products.
Contraindications	<ul style="list-style-type: none"> Do not use on denuded, excoriated or broken skin as some components of the product may be irritating.
Formats & Sizes	<ul style="list-style-type: none"> Tube <ul style="list-style-type: none"> 28g 92g
	
Application Directions	Rationale
Skin should be clean and dry prior to application. Apply cream SPARINGLY to cover entire area.	In order for the cream to work effectively, the skin should not feel oily, or have a white residue after application.
To Remove	
Removal is not necessary prior to reapplication.	
Frequency of Dressing Change	
Reapply every 2-3 days as needed. If cream is being used under wound dressings, re-apply with each dressing change. If cream is being used for incontinence protection then apply cream sparingly to affected areas as above. Apply cream once a day.	Dressing may remove cream protection with its removal. If patient is incontinent greater than 3-6x/day, then application of cream will be required more frequently.
Expected Outcome	
Skin will remain healthy and intact.	
For further information, please contact your Wound Clinician	