




Skin and Wound Product Information Sheet

Combiderm Non Adhesive	
Classification	Cover Dressing: Composite
Key Points	<ul style="list-style-type: none"> • Non adhesive absorbent pad dressing
Indications	<ul style="list-style-type: none"> • For wounds with small to moderate amounts of exudate • May be used as a primary dressing or in combination with other wound products as a secondary dressing
Precautions	<ul style="list-style-type: none"> • N/A
Contraindications	<ul style="list-style-type: none"> • Do not use on untreated clinically-infected wounds
Formats & Sizes	<ul style="list-style-type: none"> • Non-Adhesive dressing <ul style="list-style-type: none"> ▪ 8 x 8 cm ▪ 13 x 13 cm ▪ 15 x 25 cm <div style="text-align: center; margin-top: 10px;">  </div>
Application Directions	Rationale
Cleanse/irrigate wound with sterile normal saline or agency approved wound cleanser; dry peri-wound skin.	Reduces wound debris and allows for adhesion of dressing or tape.
If required, apply a skin barrier to peri-wound skin.	To protect the peri-wound skin from maceration and to improve the adhesion of the dressing or tape.
To Apply	
Choose dressing size that extends at least 3cm beyond the wound edge. Do not cut the absorbent center part.	Incorrect sizing will adversely affect the dressing function.
Remove the release paper and apply directly to the wound as a primary dressing or in combination with another wound product as a secondary cover dressing For heavily exudating wounds, use with a hydrofiber to extend wear time. Secure dressing with tape.	The dressing will swell and soften as it absorbs exudate.
To Remove	
Gently remove the tape and lift dressing off of wound.	To minimize trauma to the peri-wound skin.
Frequency of Dressing Change	
Will depend on the amount of exudate. May be left on up to 7 days.	
Expected Outcome	
Exudate is managed with no peri-wound maceration.	
For further information, please contact your Wound Clinician.	