


Skin and Wound Product Information Sheet

InterDry	
Classification	Transfer Sheet: Skin
Key Points	<ul style="list-style-type: none"> Knitted polyester fabric sheet impregnated with silver which reduces skin-to-skin friction, wicks moisture from skin surface, manages odour and provides antimicrobial action for up to 5 days. Sheet is imprinted with 5x5cm (2x2in) grid pattern to assist with selecting the correct size of fabric.
Indications	<ul style="list-style-type: none"> For the prevention and management of intertrigo (where skin touches skin, creating moisture and friction which causes redness, itching, burning, odour and/or maceration) e.g., groin, abdomen, axilla, toes, fingers, buttock. May be used to provide comfort for the client with a radiation skin reaction. May be used under devices such as compression wraps, braces, or splints. May be used for moisture management of denuded skin.
Precautions	<ul style="list-style-type: none"> Product is not to be washed. Always leave at least 5cm (2in) of the sheet outside of the skin fold. If area is to be moisturized, ensure that it is dry before applying the sheet.
Contraindications	<ul style="list-style-type: none"> Do not use for clients with known sensitivity or allergy to silver. Do not use as a cover for creams, pastes, powders, or ointments. Do not use for wounds.
Formats & Sizes	<ul style="list-style-type: none"> Roll <ul style="list-style-type: none"> 25 x 366 cm (roll) Package <ul style="list-style-type: none"> 25 x 92 cm <div style="text-align: center;">  </div>
Application Directions	Rationale
Sheet comes perforated but may be cut in whichever direction needed for a good fit in the skin fold or area.	
Application – Day 1	
For Skin Folds: <ul style="list-style-type: none"> Cleanse skin folds with no-rinse cleanser. Pat dry. Cut a length of sheet to fit the fold, add an additional 5cm (2in) or more to create a ‘tail’. Lay the sheet into the base of the skin fold, smooth it into the skin fold to remove any creases. Secure sheet into place as appropriate. Write the current date on the exposed piece of fabric. 	The ‘tails’ need to be exposed to air to allow for wicking and evaporation of moisture.
For Radiation Skin Reaction: <ul style="list-style-type: none"> Cleanse and moisturize the skin area as directed by Oncology Radiation care team. Ensure that the area is dry. Cut a length of sheet to cover the affected area. Lay the sheet over the area, smooth to remove creases. Secure the sheet into place as appropriate. Write the current date the sheet. 	
Application – Days 2, 3, 4	
Remove the InterDry sheet daily and cleanse the skin with cleanser e.g., no-rinse; pat dry. Assess skin condition. Reposition same sheet as above.	Cleanse and inspect skin daily to monitor healing. Single piece can be re-used unless soiled with urine, feces, or excessive perspiration.
Application – Day 5	
Discard the sheet and apply new sheet, if needed.	
Expected Outcome	
Intertrigo is prevented or if present, resolved within 2 weeks. Radiation Skin Reaction discomfort is minimized.	
For further information, please contact your Wound Clinician.	