




## Skin and Wound Product Information Sheet

<b>Mesorb</b>	
<b>Classification</b>	<b>Cover Dressing: Composite</b>
<b>Key Points</b>	<ul style="list-style-type: none"> <li>• Highly absorbent core</li> <li>• Fluid repellent backing which also covers the sides to prevent side leakage</li> <li>• Breathable</li> </ul>
<b>Indications</b>	<ul style="list-style-type: none"> <li>• For moderate to heavily exuding wounds which require frequent dressing changes</li> <li>• Can be used as a primary dressing or in combination with other wound products as a secondary dressing</li> </ul>
<b>Precautions</b>	<ul style="list-style-type: none"> <li>• N/A</li> </ul>
<b>Contraindications</b>	<ul style="list-style-type: none"> <li>• Do not use on untreated clinically-infected wounds</li> </ul>
<b>Formats &amp; Sizes</b>	<ul style="list-style-type: none"> <li>• Non- Adhesive Dressings                             <ul style="list-style-type: none"> <li>▪ 10 x 10 cm</li> <li>▪ 10 x 15 cm</li> <li>▪ 10 x 20 cm</li> <li>▪ 15 x 20 cm</li> <li>▪ 20 x 25 cm</li> <li>▪ 30 x 20 cm</li> </ul> </li> </ul> <div style="text-align: right;">  </div>
<b>Application Directions</b>	<b>Rationale</b>
Cleanse the wound and peri-wound skin with sterile normal saline or agency approved wound cleanser; dry peri-wound skin.	Reduces wound debris and allows for adhesion of dressing or tape.
If required, apply skin barrier to peri-wound skin.	To protect the peri-wound skin from maceration and to improve the adhesion of the dressing or tape
<b>To Apply</b>	
Apply Mesorb to the wound with at least 2-4 cm overlap on the peri-wound skin	Facilitates absorption of exudate at the edges of the wound.
May be used as a primary dressing or in combination with other wound product as a secondary cover dressing.	
Secure in place with tape or fixation device e.g. Kling and tape.	
<b>To Remove</b>	
Gently remove tape from peri-wound skin.	To minimize trauma to the peri-wound skin.
<b>Frequency of Dressing Change</b>	
Will depend upon the amount of exudate.	
<b>Expected Outcome</b>	
Exudate is managed with no peri-wound skin maceration.	
<b>For further information, please contact your Wound Clinician.</b>	