




## Product Information Sheet

OASIS Bedbath Wipes	
<b>Classification</b>	<b>Cleanser: Prepared Cloth – Maceratable</b>
<b>Key Points</b>	<ul style="list-style-type: none"> <li>• Pre-moistened, rinse-free, maceratable wipes with a skin cleanser and moisturizer.</li> <li>• Wipes break down completely, do not block human waste macerator disposal units such as Vernacare.</li> <li>• Lanolin, alcohol, latex and perfume free.</li> </ul>
<b>Indications</b>	<ul style="list-style-type: none"> <li>• For use when providing a bed bath.</li> </ul>
<b>Precautions</b>	<ul style="list-style-type: none"> <li>• For external use only.</li> <li>• May be used for cleansing the face but avoid direct contact to the eye.</li> <li>• Can be used for incontinence clean-up but recommend using products intended for this purpose (e.g., <a href="#">Remedy Foam Cleanser</a>).</li> <li>• Do not use to remove Extra Protective Cream, use <a href="#">Remedy Foam Cleanser</a>.</li> <li>• Dispose in garbage or macerator. <b>Do not flush.</b></li> </ul>
<b>Contraindications</b>	<ul style="list-style-type: none"> <li>• Sensitivity or allergy to any of the product ingredients.</li> </ul>
<b>Formats &amp; Sizes</b>	<ul style="list-style-type: none"> <li>• Package:                             <ul style="list-style-type: none"> <li>○ 5 wipes (30x 22cm)</li> <li>○ 8 wipes (30 x 22cm)</li> </ul> </li> </ul> <div style="text-align: right;">  </div>

Directions	Rationale / Key Points
<b>Selection</b>	
Package is single client use.	To minimize risk of cross contamination.
Choose size of package based on client need (e.g., package of 5 for a lighter bath).	To minimize wastage.
<b>Preparation</b>	
Use at room temperature or warm in an Oasis Warmer. Warming in microwave is not recommended by manufacturer.	Warm cloths provide increased client comfort.
Each product slot in Oasis Warmer has lights that indicate the status of the package in the slot: <ul style="list-style-type: none"> <li>• Solid red indicates that the product is being warmed.</li> <li>• Solid green indicates the product is warm and ready for uses.</li> <li>• Flashing green indicates that the product should be used first as it has been in warmer for more than 84 hours.</li> <li>• Flashing red indicates that the product should be discarded as it has been in the warmer for more than 200 hours.</li> </ul>	Wipes will warm in approximately 90 minutes.
Once package has been removed from warmer and taken to client’s bedside do not put back in warmer for re-warming.	To reduce the risk of cross-contamination between clients.
Closely monitor heat level, especially for infants or unresponsive or skin-sensitive individuals. Do not use if excessively hot.	Use caution when determining heat level as gloves diminish sensitivity to heat.
<b>Application</b>	
Use 1 washcloth for each anatomical region, for example, face, perineal area. Do not rinse.	To reduce the spread of microorganisms.
Discard each cloth into garbage after use or dispose of in macerator. <b>Do not flush.</b>	Cloths may plug/clog toilet plumbing.



## Product Information Sheet

Directions	Rationale / Key Points
<b>Expected Outcomes</b>	
Clean moisturized skin. Product performs as expected.	If product does not perform as expected, notify NSWOC/Wound Clinician and then consider submitting a <a href="#">Supply Chain Product Concern Form</a> .
<b>For further information please contact NSWOC/Wound Clinician</b>	