




## Skin and Wound Product Information Sheet

<b>PolyMem Finger/Toe Backorder Substitution for 3M Adaptic Digit</b>	
<b>Key Points</b>	<ul style="list-style-type: none"> <li>• PolyMem dressing is being used as a PHSA Supply Chain approved substitution for Adaptic Digit for two sizes; large finger and large toe.</li> <li>• PolyMem product is silicone-coated fabric tubular bandage designed for fingers and toes.</li> </ul>
<b>Indications Precautions Contraindications</b>	<ul style="list-style-type: none"> <li>• As for <a href="#">Adaptic Digit</a></li> </ul>
<b>Size and formats</b>	<ul style="list-style-type: none"> <li>• Finger/Toe circumference:                             <ul style="list-style-type: none"> <li>○ Large 6.72 – 7.74 cm (2.6-3.0")</li> <li>○ Ex-Large 7.74 – 8.76 cm (3.0-3.4")</li> </ul> </li> </ul> <div style="text-align: right;">  </div>
<b>Application Directions</b>	<ul style="list-style-type: none"> <li>• Measure tubular dressing against digit (finger or toe) to determine the length of dressing</li> <li>• Cut dressing to size.</li> <li>• Remove the insert from the rolled end of the dressing; discard insert.</li> <li>• Insert digit into the rolled end and push finger into the dressing.</li> <li>• Roll dressing onto the digit.</li> <li>• Ensure dressing fits securely and not too tight on the digit.</li> </ul>
<b>Dressing Change Frequency</b>	<ul style="list-style-type: none"> <li>• Change dressing when it becomes soiled or loose.</li> </ul>
<b>For further information, please contact your NSWOC/Wound Clinician.</b>	