














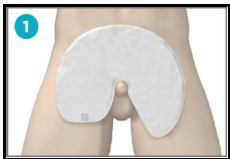

## Product Information Sheet

QuickChange Wrap	
<b>Classification</b>	<b>Continence Containment Product / Male Penile Wrap</b>
<b>Key Points</b>	<ul style="list-style-type: none"> <li>• Specially designed wrap made to encircle the penis; has a super-absorbent gel core that absorbs urine a way from the skin and an outer urine proof barrier to keep surrounding skin, linen and clothes dry.</li> </ul>
<b>Indications</b>	<ul style="list-style-type: none"> <li>• Males who are bedfast, chair-fast or minimally mobile and require a small pad continence containment product that can manage up to 500ml of urine.</li> <li>• May be used for those with a standard/longer-sized, small/retracted or hidden penis and those with scrotal edema.</li> <li>• May be used with or without mesh pant.</li> </ul>
<b>Precautions</b>	<ul style="list-style-type: none"> <li>• For those with some mobility, use the wrap with a mesh pant.</li> </ul>
<b>Contraindications</b>	<ul style="list-style-type: none"> <li>• Do not use with a brief, (e.g., pull-up, tabbed or belted).</li> </ul>
<b>Formats &amp; Sizes</b>	<ul style="list-style-type: none"> <li>• Wrap:                             <ul style="list-style-type: none"> <li>○ One size</li> </ul> </li> </ul> <div style="text-align: center;">  </div>

Directions	Rationale / Key Points
<b>Selection</b>	
For most applications, one wrap is most often used, but two wraps will be needed for a penis hidden by a large abdomen.	
<b>Preparation</b>	
Cleanse the peri-area.	
Apply skin protectant to area as per care plan.	
<b>Application</b>	
<p>For standard or long penis:</p> <ul style="list-style-type: none"> <li>• Position the penis on the lower abdomen with penis facing straight towards the head.</li> <li>• Slide wrap under the penis (Step1) with:                             <ul style="list-style-type: none"> <li>○ White side up</li> <li>○ Round edge towards the head</li> <li>○ The flaps (straight &amp; pointed) towards the legs.</li> </ul> </li> <li>• Fold the pointed flap over top of the penis at a 45° angle (Step2); bring flap edge up to meet the rounded top edge.</li> <li>• Fold the straight flap up over the wrap; slightly pull the flap upward to align its edge with the round edge (Step3).</li> <li>• Using two fingers, pinch the hook fastener and inner pointed flap together. If needed, secure round edge with small piece of tape.</li> <li>• Lay the wrap flat against the lower abdomen.</li> </ul>	<div style="display: flex; flex-direction: column; align-items: center;"> <div style="margin-bottom: 10px;">  </div> <div style="margin-bottom: 10px;">  </div> <div>  </div> </div>
<p>For small or retracted penis:</p> <ul style="list-style-type: none"> <li>• Slide wrap under the penis (Step1) with:                             <ul style="list-style-type: none"> <li>○ White side up</li> <li>○ Round edge towards the head</li> <li>○ The flaps (straight &amp; pointed) towards the legs.</li> </ul> </li> <li>• Slide the pointed flap underneath the scrotum.</li> </ul>	<div style="display: flex; flex-direction: column; align-items: center;"> <div>  </div> </div>



## Product Information Sheet

Directions	Rationale / Key Points
<ul style="list-style-type: none"> <li>Lift the flap and scoop scrotum forward.</li> <li>Pull the straight flap over and under the pointed flap (Step2).</li> <li>Using two fingers, pinch hook fastener and the inner pointed flap together. If needed, secure rounded edge with small piece of tape.</li> <li>If possible, lay the wrap on the lower abdomen (Step3.)</li> </ul>	 
<p>For scrotal edema:</p> <ul style="list-style-type: none"> <li>Slide wrap under the penis (Step1) with:                             <ul style="list-style-type: none"> <li>White side up.</li> <li>Round edge towards the head.</li> <li>The flaps (straight &amp; pointed) towards the legs (Step2).</li> </ul> </li> <li>Slide the pointed flap underneath the scrotum.</li> <li>Pull the straight flap over and position it under the pointed flap to create a bowl (Step3) which will cradle the edematous scrotum.</li> <li>Using two fingers, pinch the hook fastener and the inner pointed flap together.</li> </ul>	  
<p>For hidden penis (need two wraps):</p> <ul style="list-style-type: none"> <li>If needed, lift abdominal skin-fold up to see peri-area.</li> <li>Slide the first wrap around scrotum (Step1) with:                             <ul style="list-style-type: none"> <li>White side up.</li> <li>Round edge towards the feet.</li> <li>The flaps (straight &amp; pointed) towards the head.</li> <li>Tuck the wrap between the thighs.</li> </ul> </li> <li>Place the second wrap (Step2):                             <ul style="list-style-type: none"> <li>White side up over the belly.</li> <li>Position the wrap to ensure the penis opening is visible.</li> <li>Fold edge of straight flap over to cover/hide fastener.</li> <li>Fold down the wrap lengthwise to cover the penis opening (Step3).</li> </ul> </li> <li>If needed, lay abdominal skin-fold down over the peri-area.</li> </ul>	  
<b>Removal</b>	
<p>Where one wrap is used; undo the fastener and remove wrap. Where two wraps have been used; remove the wraps.</p>	
<b>Frequency of Wrap Change</b>	
<p>Check wrap with every two hours to ensure wrap is in the correct position.  Change wrap when wet.</p>	<p>Client movement may move the wrap out of position</p>
<b>Expected Outcomes</b>	
<p>Urine is contained within the wrap protecting skin integrity. Product performs as expected.</p>	<p>If product does not perform as expected, notify NSWOC/ Wound Clinician and then consider submitting a <a href="#">Supply Chain Product Concern</a> form.</p>
<p><b>For further information please contact NSWOC/Wound Clinician</b></p>	