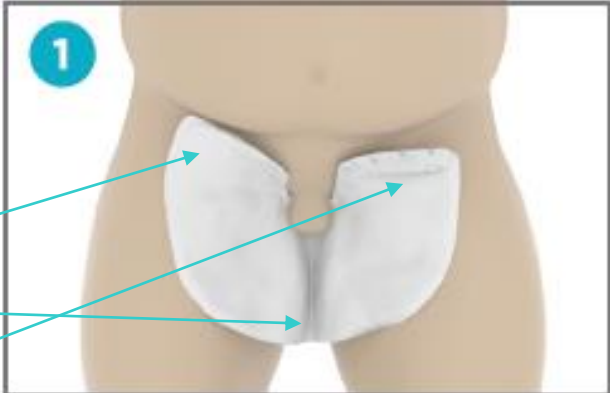
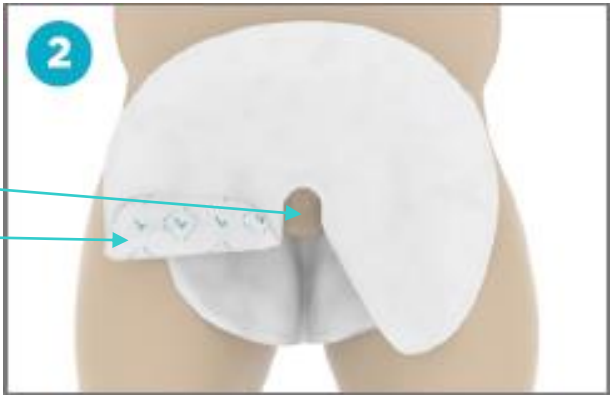


QuickChange Male Penile Wrap Application for a Hidden/Covered Penis

<p>Step 1: Place the 1st pad</p> <ul style="list-style-type: none">• Apply skin protectant to area as per care plan.• If needed, lift abdominal skin fold up to see peri-area.• Slide pad around scrotum with:<ul style="list-style-type: none">◦ White side up, round edge towards the feet, tuck the wrap between the thighs.◦ Pointed flap and◦ Straight flap towards the head. <p>Note: straight flap has hook fastener, fold over the top of flap to hide this.</p>	
<p>Step 2. Place the 2nd pad</p> <ul style="list-style-type: none">• Place white side up over the belly.• Position the wrap to ensure the penis opening is visible.• Fold over the edge of straight flap to cover the fastener.	
<p>Step 3. Position the pad</p> <ul style="list-style-type: none">• Fold 2nd wrap in half lengthwise and cover the penis opening.• If needed, lay the abdominal skin fold back down over the peri-area.	