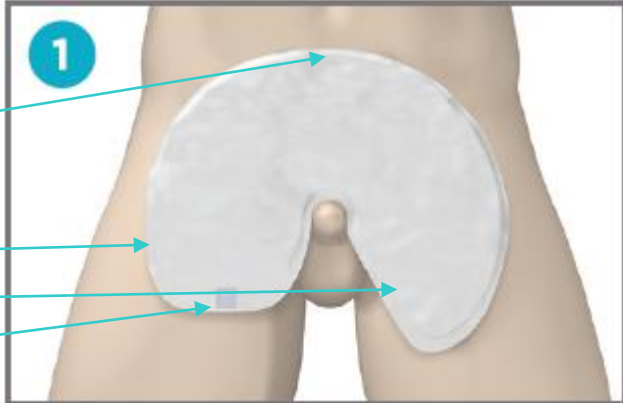



QuickChange Male Penile Wrap

Application for Scrotal Edema

<p>Step 1: Place the pad</p> <ul style="list-style-type: none">• Apply skin protectant to area as per care plan.• Slide pad around the penis with:<ul style="list-style-type: none">○ White side up, round edge towards the client's head○ Straight flap and○ Pointed flap towards the legs. <p>Note: straight flap has hook fastener.</p>	
<p>Step 2. Surround the penis/scrotum</p> <ul style="list-style-type: none">• Slide pointed flap underneath the scrotum.• Pull the straight flap over and position under the straight flap to create a bowl to cradle the edematous scrotum.	
<p>Step 3. Secure the wrap</p> <ul style="list-style-type: none">• Using two fingers, pinch the hook fastener and inner pointed flap together.	