




Skin and Wound Product Information Sheet

Unna-Z Backorder Substitution for S&N Viscopaste	
Key Points	<ul style="list-style-type: none"> • Medline product - zinc oxide impregnated open weave gauze wrap. • Compared to Viscopaste: <ul style="list-style-type: none"> ○ May contains slightly more zinc oxide; 10-16% vs 10%. ○ Width of wrap is slightly wider; 10cm vs 7.5cm. ○ Length of roll is longer 9m vs 5.5m. • Preservative free. • Natural rubber latex free.
Indications Precautions Contraindications	<ul style="list-style-type: none"> • As for Viscopaste .
Size and formats	<ul style="list-style-type: none"> • Gauze wrap 10cm x 9m. <div style="text-align: center;">  </div>
Application Directions	<ul style="list-style-type: none"> • Position the foot at a right angle to the leg (dorsiflexion). • Start at the base of the toes, using no tension, and an overlap of 50%, loosely wrap the bandage around the foot, heel, and ankle; ensure all areas are covered. • Starting above the ankle, using no tension, and overlapping each turn up the leg by 50%, wrap the bandage up the leg using a spiral technique. • Stop two finger widths below the knee; cut off excess wrap. • Smooth the wrap to remove any wrinkles and mold it to the shape of the leg. • Wrap the leg with Kling in a loose spiral and secure with tape or mesh stockinet – this will protect client’s clothing/environment from the zinc.
Dressing Change Frequency	<ul style="list-style-type: none"> • Change at least once a week or when wrap slips.
For further information, please contact your NSWOC/Wound Clinician.	